

Dear Doctor, Please find mentioned quality specifications and guidelines for elastomeric impressions required by ClearPath.

- 1) **Elastomeric Impression** to be taken in **two-step technique**
- 2) **BLOCK DEEP UNDERCUTS** before taking impressions
- 3) **All dental work should be completed** before taking impressions for ClearPath
- 4) **NO tray exposure** should be there at cuspal heights or incisal edges
- 5) **EXCESSIVE** putty show through should not be present
- 6) All details **TILL THE LAST MOLAR** in all quadrants should be recorded
- 7) **No breakage of inter-dental margins** especially in the lower anterior region
- 8) No presence of “pull away” of heavy body from tray
- 9) Review the Timing, Moisture Control, Proper Tray Selection and Technique
- 10) When in doubt, repeat the impression and send multiple impressions

11) Please use local made inexpensive metallic trays as they are kept till end of treatment for records and wont be sent back with the aligners

12) Even for Single arch case, both the impressions are required

The impression should be perfect and flawless. Quality and fit of aligners depend solely on impression quality and details recorded.

Regards,
ClearPath Production

IDEAL IMPRESSION

